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## DYNAMICS THAT MOVE YOU

### Functional training with spring-loaded intelligence

The elastic bands of 4D PRO turn every movement into an experience. They give when you need power and support you when you're looking for stability. The result is a dynamic no classic suspension trainer can offer – a unique training feel that challenges, protects and motivates all at once. Physiotherapists, trainers and athletes love this springy effect because it gets the body moving intuitively and makes training noticeably more intense. Different Fazer shapes enable a more targeted, specific and deeper treatment than is possible with bare hands.

Created by Dr Dr Homayun Gharavi – a visionary who reimagined functional training to be more natural and smarter – 4D PRO pairs scientific rigour with a genuine passion for movement. Together, ARTZT thepro and 4D PRO bring back the original: backed by a strong promise of quality, professional expertise and a network that has stood for real training innovation for decades.

A suspension trainer that's more than a tool: it's the feeling of finally training exactly the way your body needs – dynamic, safe and authentic. The original – rethought.

## ASSEMBLED 4D PRO

### COMPONENTS

The 4D Pro comprises the following components:

- 1 x tree strap
- 1 x carabiner (large)
- 2 x length-adjustable, non-elastic elements
- 2 x handles
- 4 x elastic Lambda bands with carabiner and loop
- 2 x neoprene loops

# FLOOR EXERCISES

## FOR ALL FLOOR-BASED EXERCISES:

- Generally use a total of two Lambdas: one on each side (1:1), or both Lambdas on one side so both legs are in a single loop (2:0).
- Lighter and smaller users can set the 4D PRO lower (roughly knee height) and/or place the legs further into the loops, closer to the torso, to allow the spring-like closure.
- Heavier and taller people should take care not to over-stretch the bands. To prevent this, in 2:2 mode the spring effect can be increased by attaching the neoprene loops closer to the torso.
- Acceleration (the bounce) always comes from the centre of the torso, not from the limbs.
- Stretching exercises in 2:2 mode.

## 1. PELVIC LIFT

[2 x 30 sec]

- Lie on your back
- Place your lower legs in the loops
- Press your arms flat into the floor beside your body
- Lift your pelvis so your torso and legs form a straight line
- Now accelerate the hips down and up to create a swing
- Keep moving in time with the 4D PRO.

## 2. SIDE SWING

[2 x 30 sec]


- From lying on your back, open the legs and rotate to the side into a forearm support
- Brace the top arm in front of the body as shown for good force transfer
- Keep the top leg back in a long lunge position at all times
- Load both legs dynamically
- Create the swing: let the pelvis drop and accelerate upwards, drawing in the side abdominal muscles.

## 3. SPHINX

[2 x 30 sec]

- From a seated position, place your lower legs in the loops
- Cross your legs and roll over onto your front
- Forearm support, hands in fists or palms facing down, lift the hips
- Now create the swing: let the pelvis drop, then accelerate back up.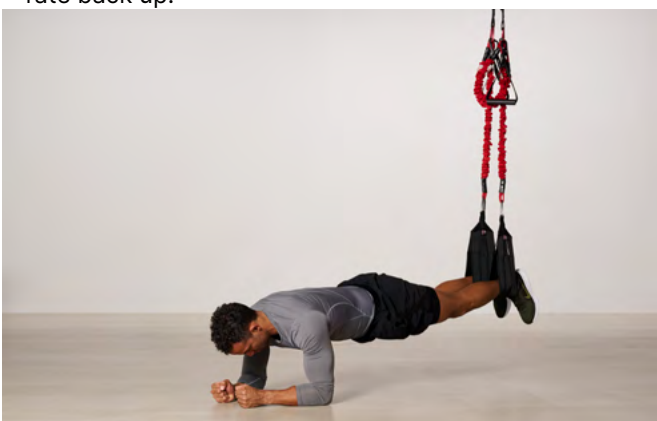

## 4. BOUNCED CRUNCH

[2 x 30 sec]


- Get into the "Sphinx" position
- Initiate the swing
- Alternate one swing with legs extended and one in a tucked position
- Knees do not touch the floor.

# STANDING EXERCISES

## FOR ALL STANDING EXERCISES:

- Start with just one Lambda per side (1:1). Progress to two bands per side (2:2). Try each exercise once asymmetrically as well (1:2/2:1).
- Arms into the loops
- Familiarisation 1: Walk forwards and backwards
- Familiarisation 2: Keep your foot position fixed. Lean lightly into the bands and maintain basic tension through the body (don't hinge at the pelvis)
- Move your feet back or forwards and, with a drive from the hips, set the bands swinging.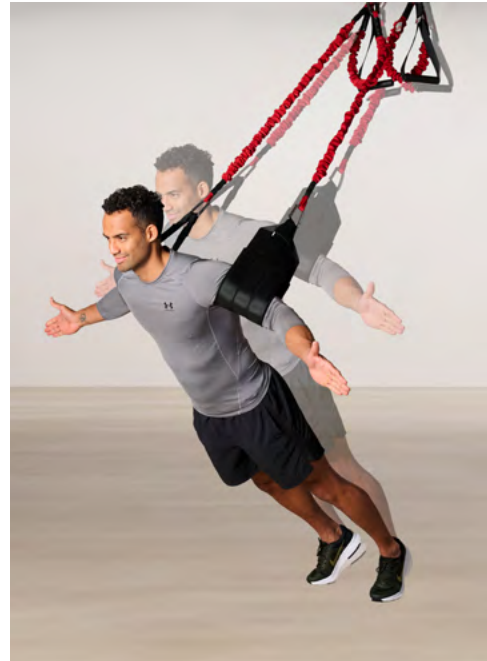

## 2. EAGLE

[2 x 45 sec]


- As in Exercise 1, but with the arms stretched out to the sides like wings
- Palms face forwards.

## 1. FALL IN

[2 x 45 sec]

- Upper arms in the loops
- Hands on your stomach
- Arms close to the body
- Step forwards
- Lean into the bands
- Step feet back and create a swing from the pelvis
- Press heels into the floor.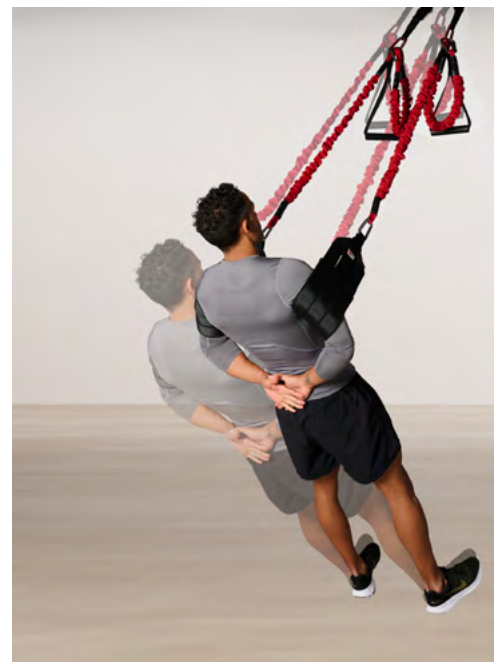

## 3. FALL BACK

[2 x 45 sec]

- Upper arms in the loops
- Hands behind the glutes
- Walk back until the band becomes taut
- Lean into the 4D PRO
- Place feet forwards, eyes on the suspension point
- Generate momentum from the hips and accelerate back to standing.

## SUSPENSION OPTIONS

### **A** with ceiling hook

If you already have a ceiling hook, make sure it has a load capacity of at least 200 kg.

Adjust the length of the non-elastic elements (3.) so that the loops (6.) hang just above knee height.

### **B** Tree strap

**Minimum load capacity of the suspension: 200 kg**

Take the tree strap (1.) and place it around the branch or bar you want to train on. Feed the loop at one end of the strap through the loop at the other end. Then clip the carabiner (2.) into the free-hanging loop.


## TRAINING TIPS

To keep enjoying your 4D PRO for a long time, always choose a body angle that prevents the bands from stretching to their maximum.


The elastic lambda bands are wear parts. That's why we recommend replacing the bands regularly, so all bands are used equally often.

Another tip: start your training slowly and build up the intensity step by step.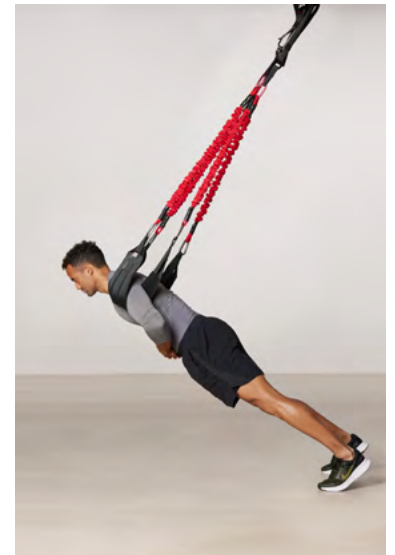

## 4D PRO COURSE

**Take it to the next level:  
4D PRO e-learning  
with Dr Dr med. Homayun Gharavi**

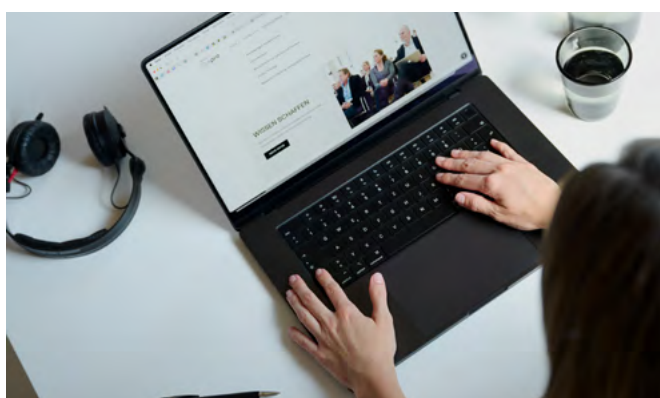**In the online course with almost 4 hours of video content, you'll get:**

- Step-by-step set-up guide incl. error check
- Theory background & core exercises
- Dynamic flows & creative workouts
- Add-ons with tips for fitness, rehab and group training

**Find more information ABOUT THE ONLINE COURSE here.**

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**Ludwig Artzt GmbH sport- és egészségmegőrző termékek**